

HOW TO T3+ DEFAULT LEAD LOCATIONS

1. Cable Bundle

F4: **ORANGE** (Right forehead EEG)
 E1: **BLACK** lead with **BEIGE** connector (Left Eye EOG)
 E2: **BLACK** lead with **BEIGE** connector (Right Eye EOG)
 PGND: **GREEN** (Patient Ground/Middle Forehead)
 M1: **GREY** lead with **BEIGE** connector, labelled "L" at both ends (Behind Left Ear)
 M2: **GREY** lead with **BEIGE** connector, (Behind Right Ear)

2. Chin EMG

1: **WHITE** lead (EMG 1)
 F: **WHITE** lead (EMG 2)

3. Leg and Chest Leads

LM1: **GREY** Long dual lead with key hole connector (Leg)
 ECG: **ORANGE** Dual lead with key hole connector (ECG Chest *Longer left)

4. Nasal Cannula and Filter

PRES: Filter plugs directly into the PRES port

