


HOW TO INDEX (AND OFFSET) NOXTURNAL VIDEO FILE

1. Open the desired study (File Explorer)

Open the desired study in windows file explorer by right clicking the study in the Noxturnal library and selecting "View in File Explorer".

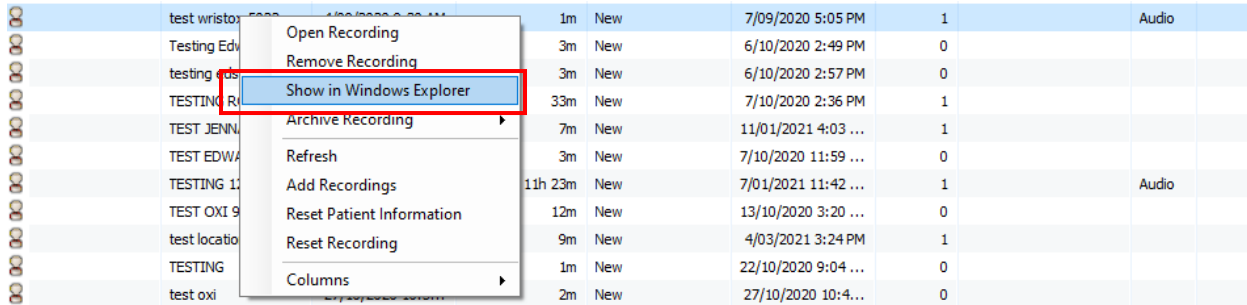

2. Copy & Remove the video file from the Study:

Within the study location, take a copy of the video file (.wmv file) and place this copy on the desktop.

Delete the file from the study location.


	thorax rip.ndf	17/05/2021 1:22 PM	Noxturnal	1,248 KB
	Video.idx	18/05/2021 1:33 PM	SQL Server Replica...	1 KB
	Video_00_000.wmv	18/05/2021 1:33 PM	WMV File	335,509 KB
	voltage (battery).ndf	17/05/2021 1:22 PM	Noxturnal	4,986 KB
	voltage (bluetooth).ndf	17/05/2021 1:22 PM	Noxturnal	4,986 KB

3. Open the Study in Noxturnal and Import the Video

Open the study in Noxturnal.

Navigate to File -> Import -> "Import Video.."

TEST TEST (Tuesday, 7 April 2020) - Noxturnal

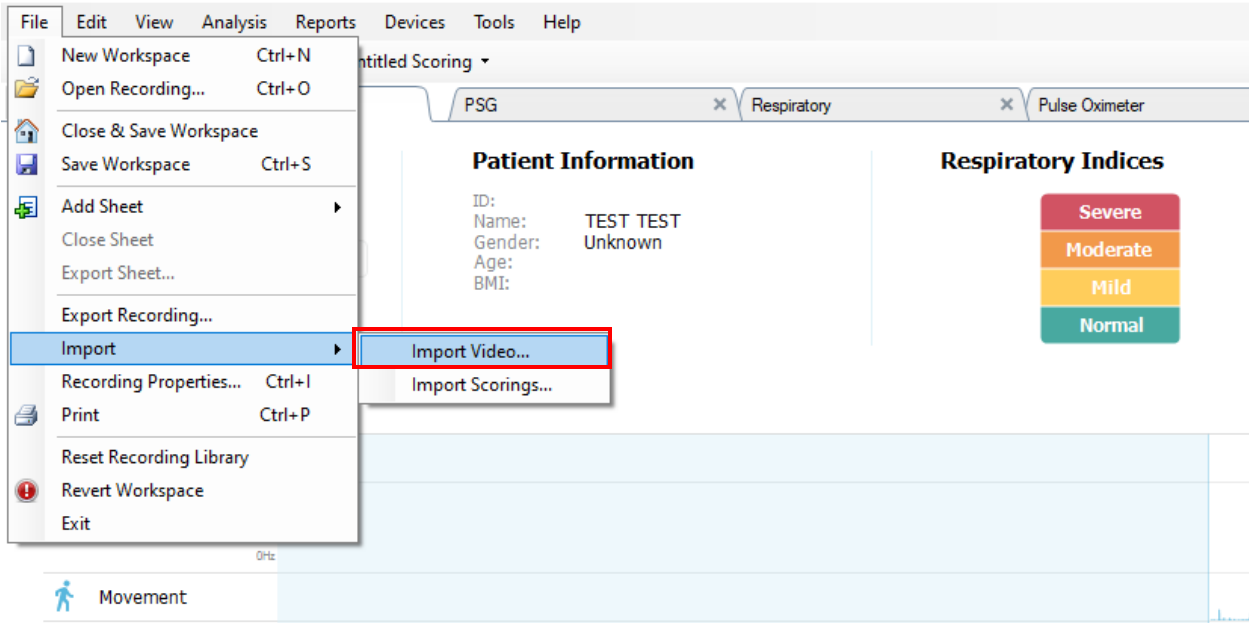

4. Select the Video File &

Import the video by selecting “Add File...”, select “Create a New Video Channel” and select Next.

Determine the import process by selecting either:

- Fixed Time (this will begin the video file at the start of the study)
- Offset from Recording at Start time (this can offset the video as required, in seconds).

