



HOW TO: REMOVE & ADD NONIN OXIMETER TO ONLINE ROOMS


Step 1: Click on "Devices (menu item)" in "Noxturnal" and select "Online Rooms" in Drop Down Menu

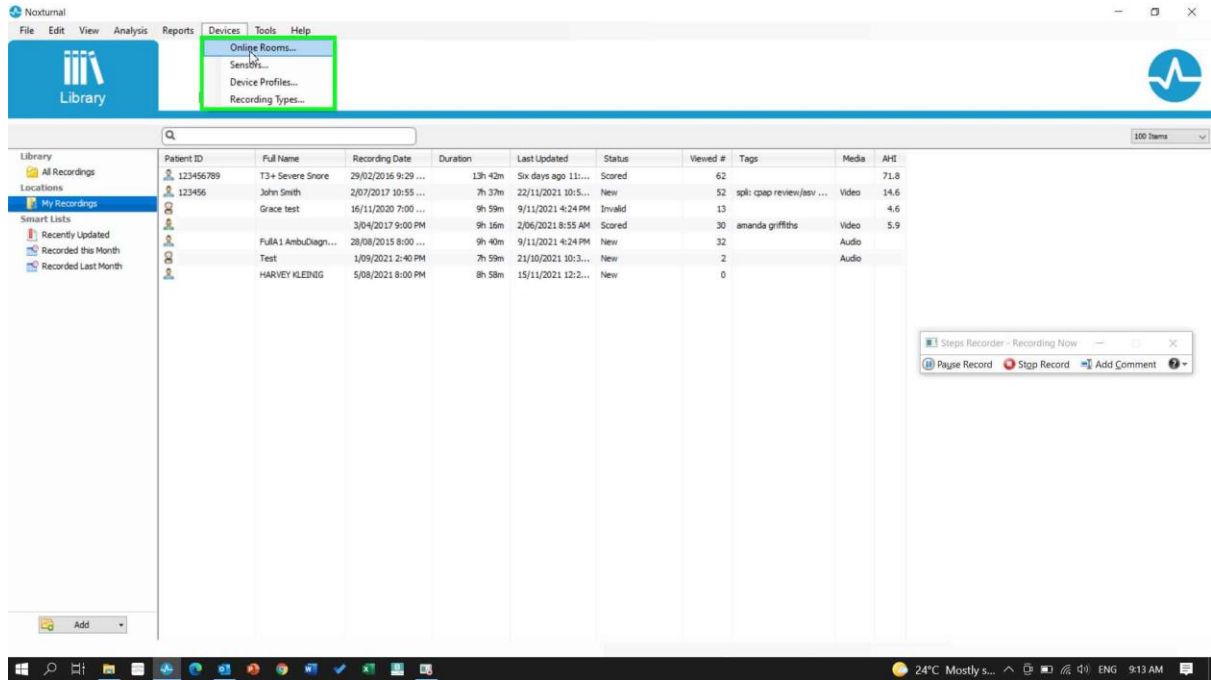


Step 2: Click on the Nonin 3150 Oximeter Icon (that you no longer wish to use) and click the "Remove Device" button.

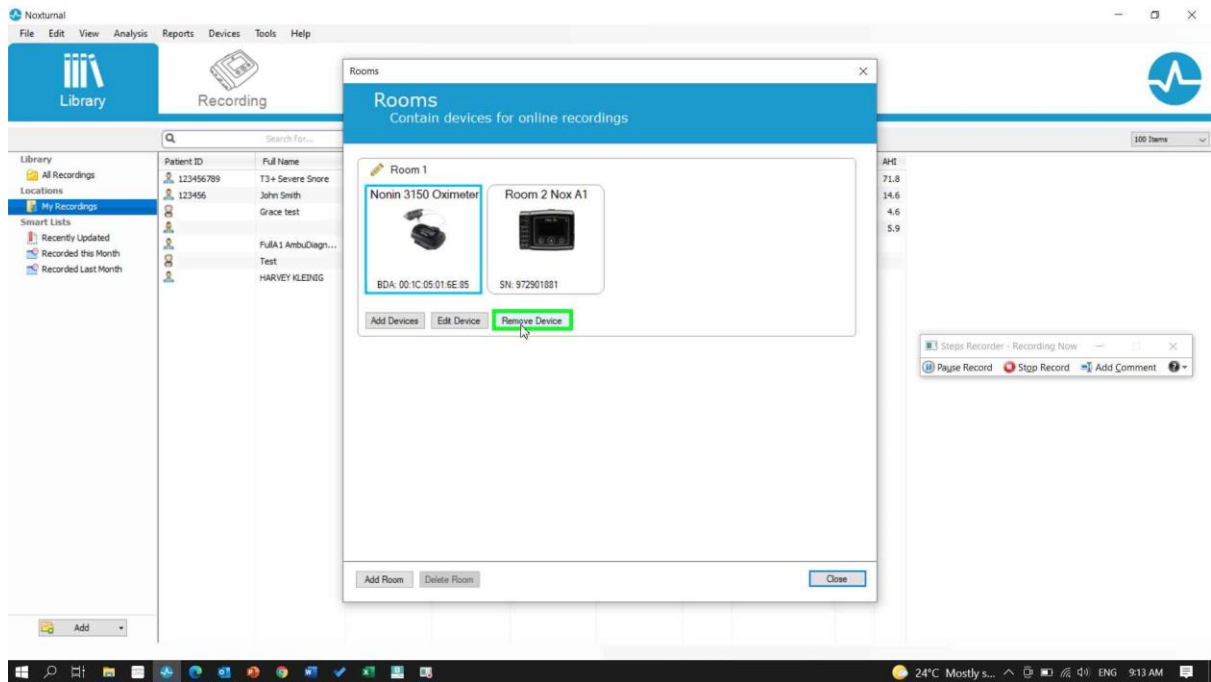


Step 3: Click on "Yes (button)" in "Warning"

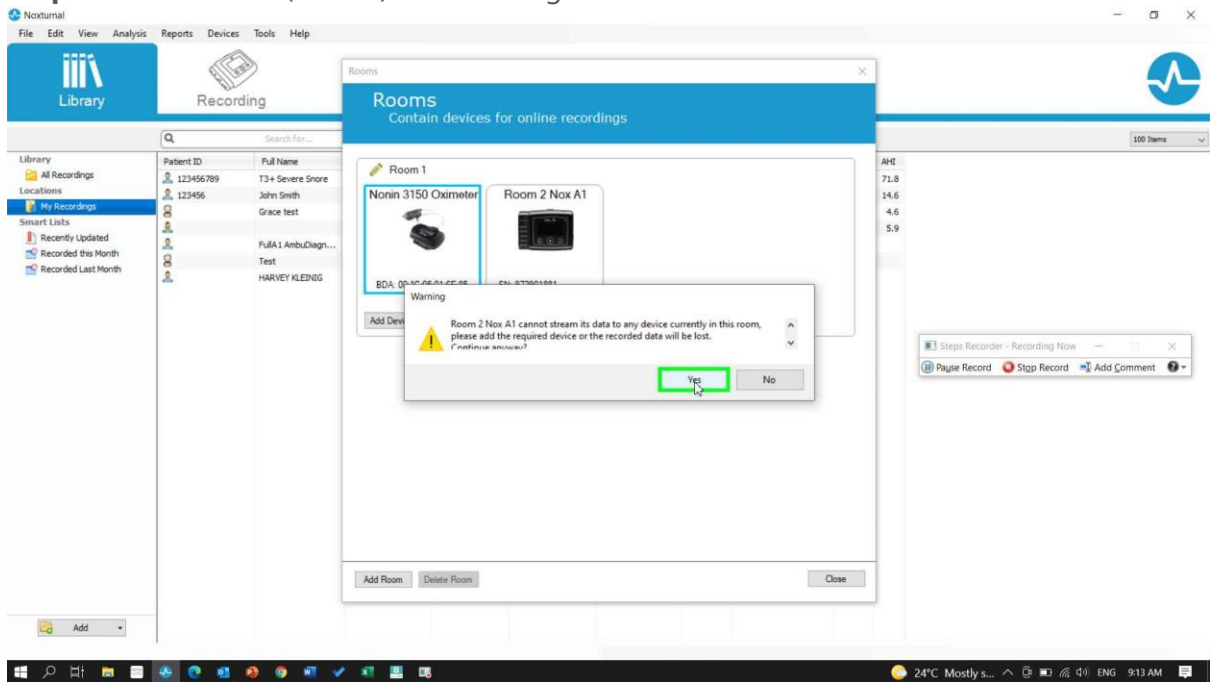


Step 4: Click on "Add Devices (button)"

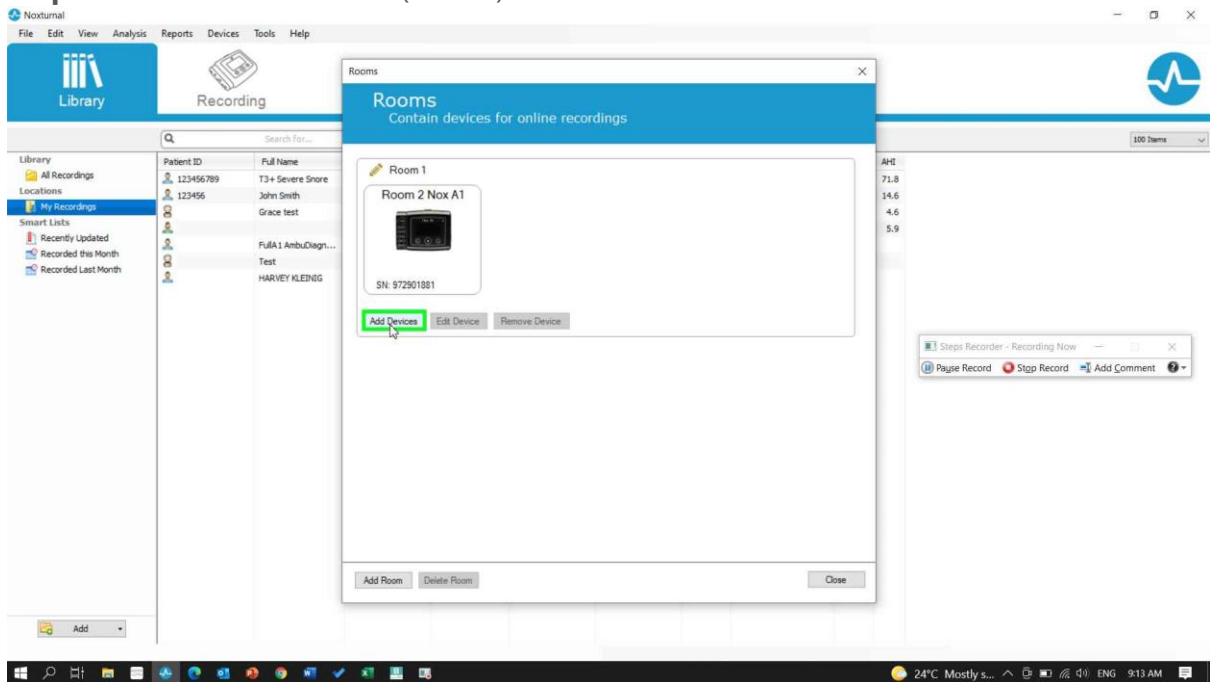


Step 5: Click on your old Nonin 3150 Oximeter in Available Devices and click on "Remove Device (button)"

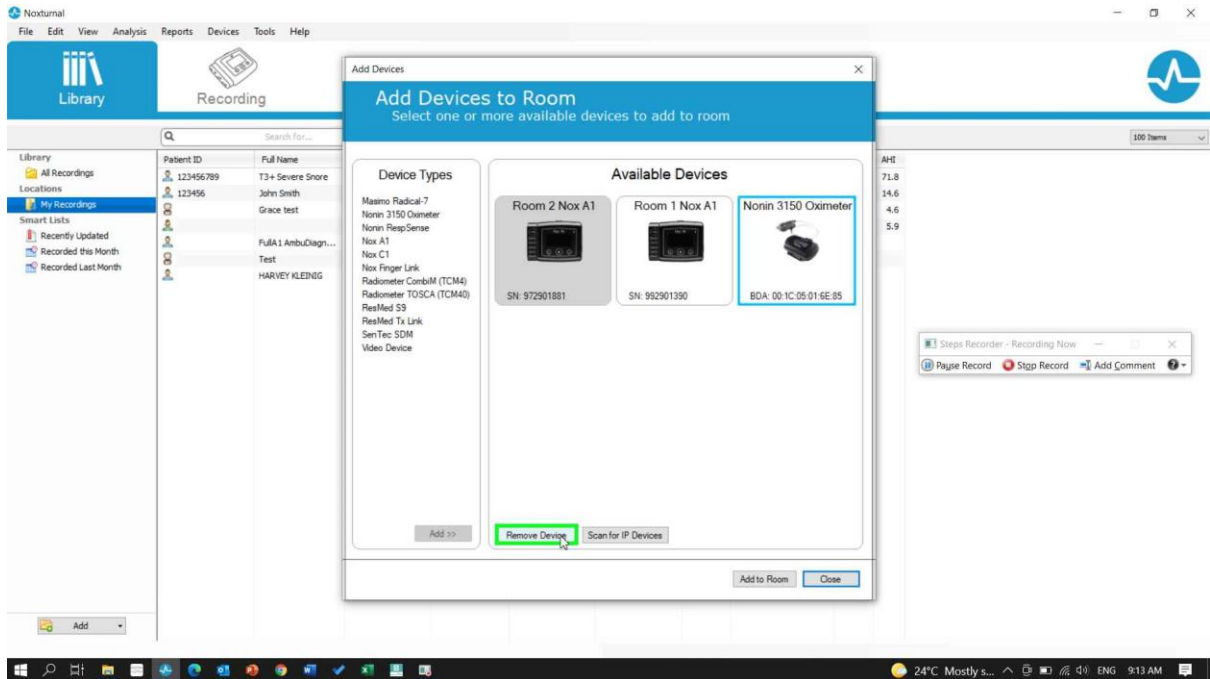


Step 6: Click on "Yes (button)" in "Remove Device"

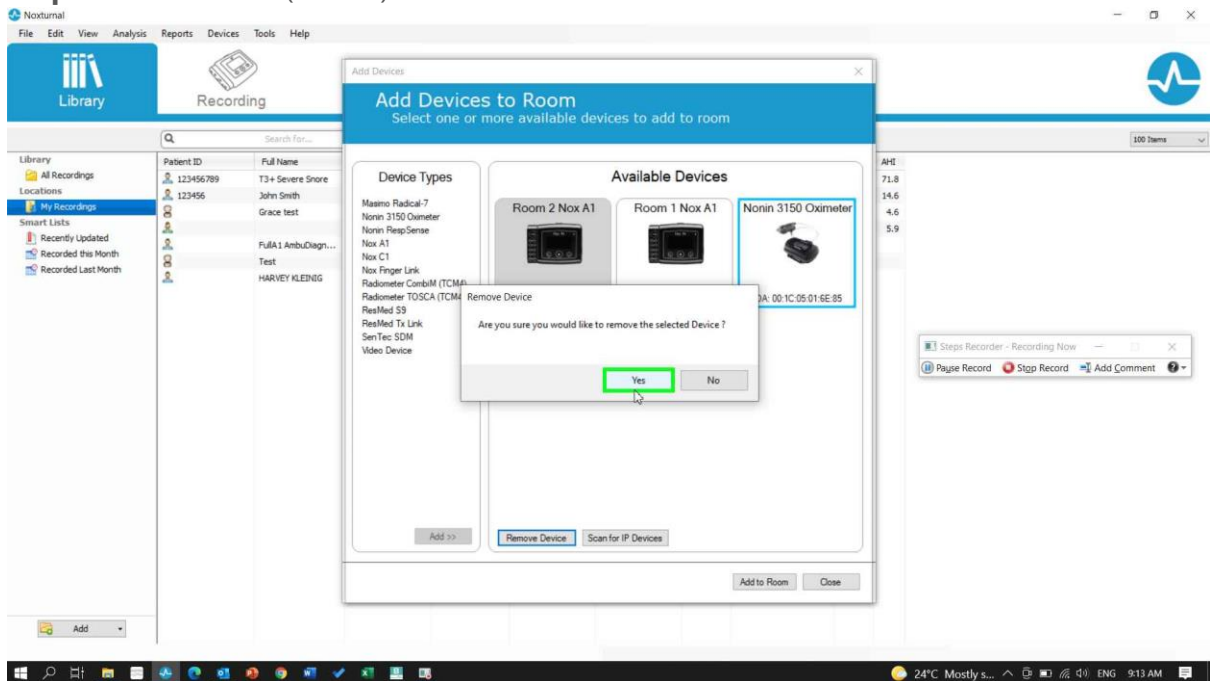


Step 7: Click on "Nonin 3150 Oximeter (text)" in Device Types to the left and click on the "Add >> (button)"

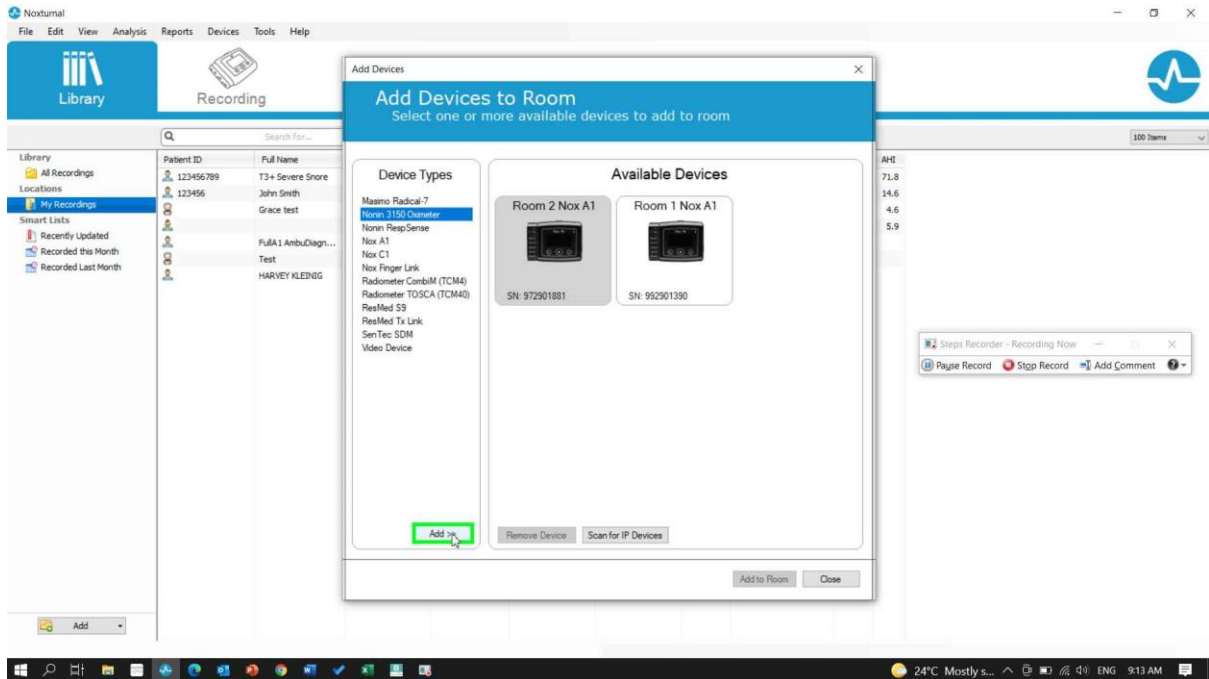


Step 8: Use the keyboard to input your New Nonin Oximeter's BDA number found on the BACK of the device. TRIPLE CHECK that this number is correct otherwise the device WILL NOT connect to your A1. Press Finish when correct.

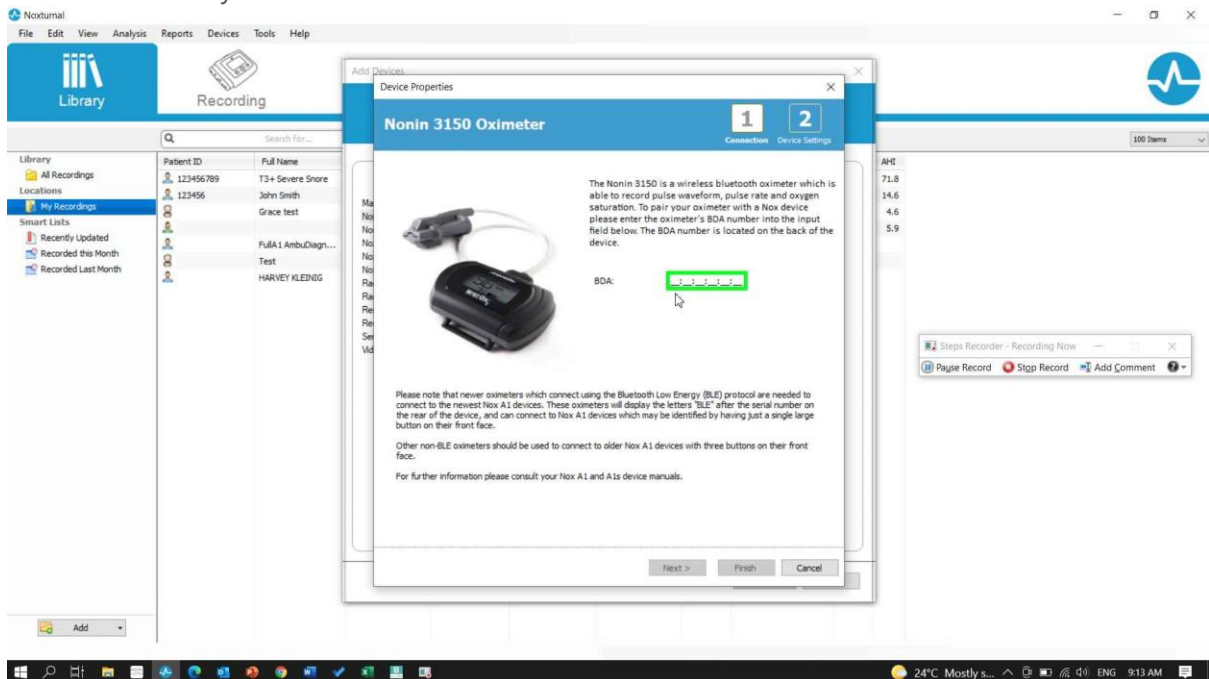

Step 9: Click on your new Nonin Oximeter and select "Add to Room"

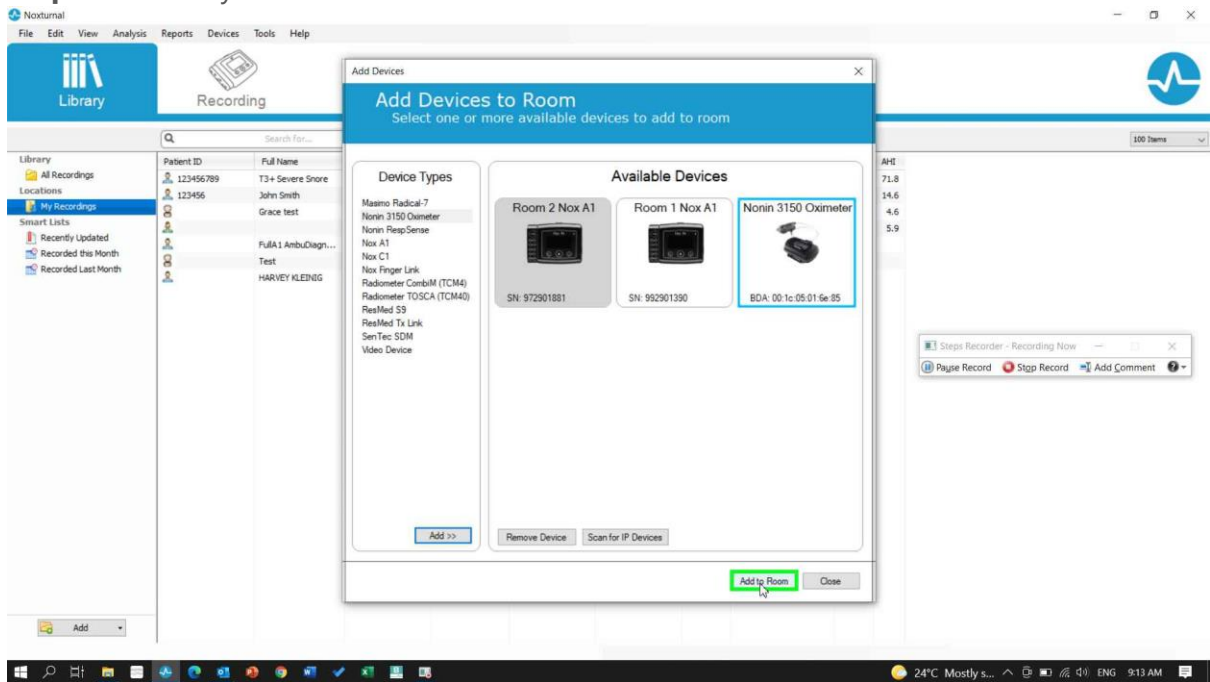

Step 10: Click on "Yes (button)" in "Warning" and then close the room.

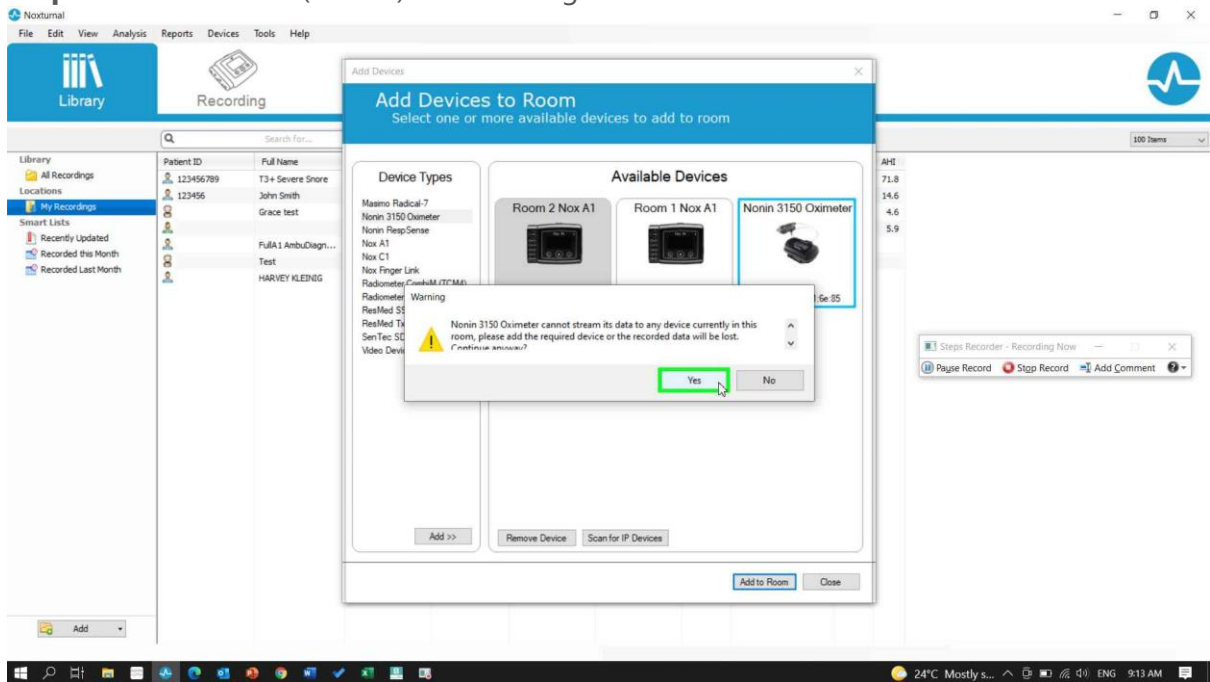

Your new Nonin Oximeter has now been added to your room. To use another oximeter, you will need to repeat these steps.