


## HOW TO: Troubleshoot A1s/T3s Oximeter pairing fails during configuration

If the oximeter pairing process fails during the configuration, you can follow the instructions below:

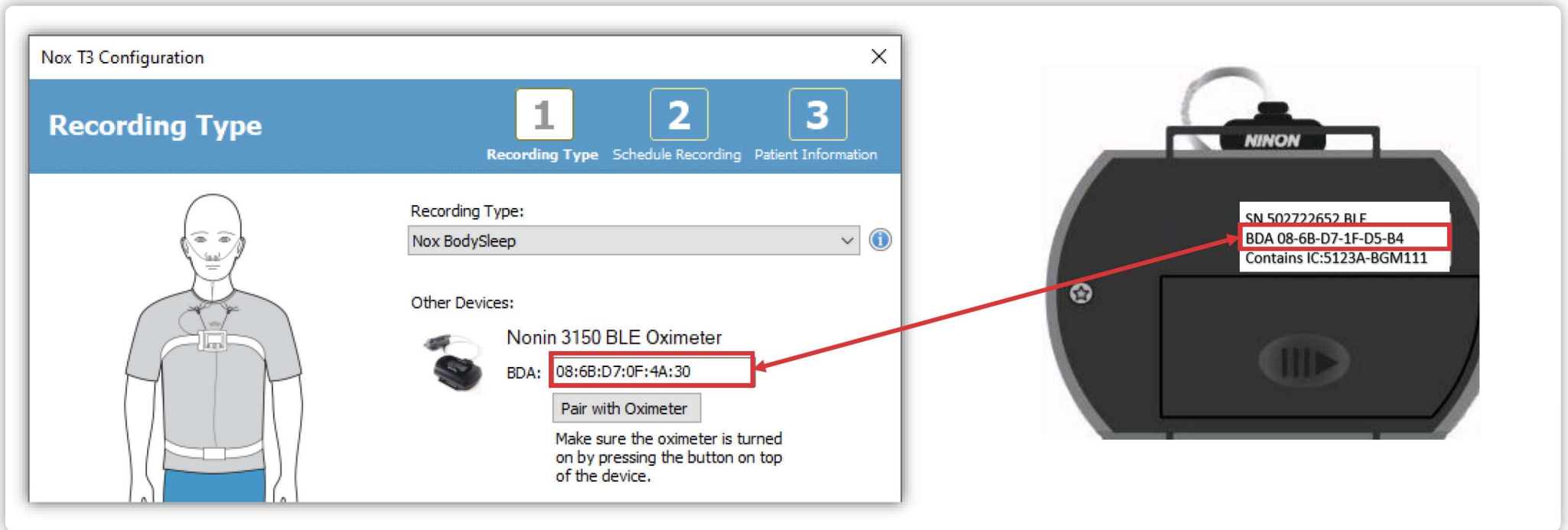

The image shows a screenshot of the 'Nox T3 Configuration' software window on the left and a photograph of the 'Nonin 3150 BLE Oximeter' on the right. A red arrow points from the BDA number in the software window to the BDA number on the oximeter's label.

**Software Window Screenshot:**

- Title: Nox T3 Configuration
- Recording Type: Nox BodySleep
- Other Devices: Nonin 3150 BLE Oximeter
- BDA: 08:6B:D7:0F:4A:30
- Pair with Oximeter button
- Instruction: Make sure the oximeter is turned on by pressing the button on top of the device.

**Oximeter Label:**

- SN 502722652 BLE
- BDA 08-6B-D7-1F-D5-B4
- Contains IC:5123A-BGM111

1. Check the BDA number in the Configuration window and make sure it matches the number on the oximeter.
2. Restart the oximeter by removing the batteries and inserting batteries again to the oximeter. Then press the grey activation button to turn on the device. Resume the pairing process.
3. Restart the Nox T3s by unplugging the Nox T3s from the computer and plugging it back in again. Resume the pairing process.