

HOW TO T3++ (A1S) DEFAULT LEAD LOCATIONS

1. Cable Bundle

- F4: **ORANGE** (Right forehead EEG)
- E1: **BLACK** lead with **BEIGE** connector (Left Eye EOG)
- E2: **BLACK** lead with **BEIGE** connector (Right Eye EOG)
- PGND: **GREEN** (Patient Ground/Middle Forehead)
- M1: **GREY** lead with **BEIGE** connector, labelled "L" at both ends (Behind Left Ear)
- M2: **GREY** lead with **BEIGE** connector, (Behind Right Ear)

2. Chin EMG


- 1: **WHITE** lead (EMG 1)
- F: **WHITE** lead (EMG 2)

3. Leg and Chest Leads

- LM1: **GREY** Long dual lead with key hole connector (Leg)
- ECG: **ORANGE** Dual lead with key hole connector (ECG Chest *Longer left)

4. Nasal Cannula and Filter

PRES: Filter plugs directly into the PRES port

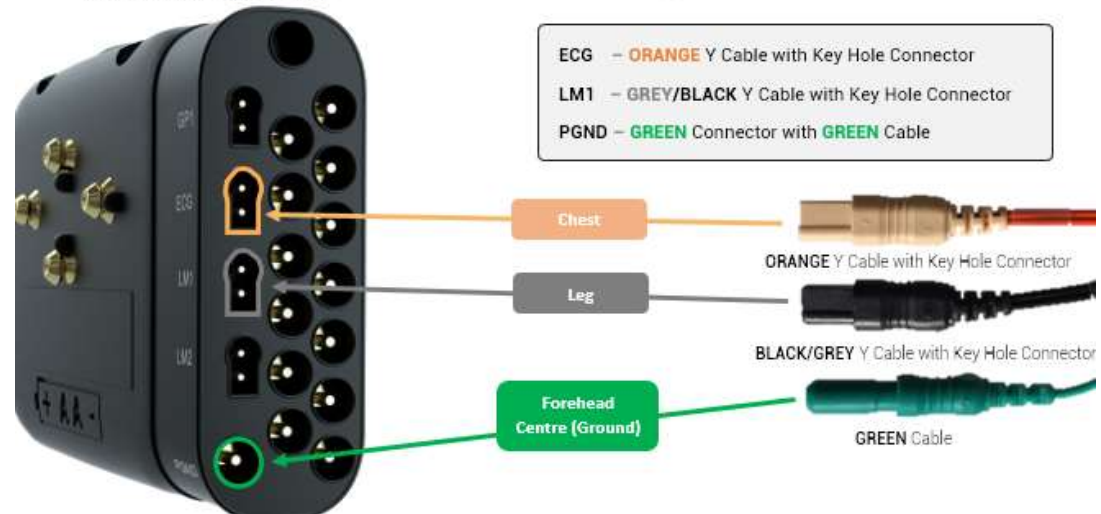

Temple Healthcare 1300 162 694